

Liver - Cholangiocarcinoma

1. SCOUT
2. CORONAL TRUEFISP BH LOCALIZER
3. AXIAL T2 HASTE MBH
4. CORONAL T2 HASTE MBH
5. AXIAL IN and OPPOSED PHASE T1 GRE BH
6. AXIAL DIFFUSION – b values of 50, 400, 800
7. CORONAL LOCALIZER non-BH
8. AXIAL T2 BLADE FS resp nav
9. T2 SPC MRCP non-BH (navigator) – cover entire liver
10. HASTE MRCP – radial stack
11. AXIAL VIBE

Inject contrast

12. AXIAL VIBE – late arterial phase (immediate) – subtraction
13. AXIAL VIBE – 70 sec.
14. CORONAL VIBE – 2 min.
15. AXIAL VIBE – 3 min.
16. AXIAL VIBE – 5 min. – subtraction
17. AXIAL AND CORONAL TRUE FISP
18. AXIAL AND CORONAL VIBE – 10 min.